

Cleaning and Maintenance Guide

To keep your Nemschoff products looking their best, please use the following tips when cleaning.

Nemschoff Healthcare Wood Finish

- ✓ Clean with a damp cloth or mild detergent solution.
- ✓ We recommend full-strength or diluted glass cleaner products since they do not leave deposits that lead to surface smearing or haze. A few examples of alternative cleaners include: Bleach, Hi-Tor® Plus, Sani-Master, Ammonia, Sursan, Matar®, TOR® HB, Quat Disinfectant, Virox™, and Virex® CaviWipes™. These cleaners must be used according to the manufacturer's instructions, and any deviation from the manufacturer's instructions may void the warranty.
- ✗ Avoid chemical and/or abrasive cleaning compounds.
- ✗ For best results, we do not recommend brand-name furniture polishes. While they will not damage the surface, they are likely to cause smearing and haze and are therefore not recommended. Products free of wax and silicones give the best long-term results and will not create problems should the surface require refinishing or repair at a future date.

Laminate Surfaces and Powder Coating

- ✓ Clean with a damp cloth or mild detergent solution.
- ✓ A few examples of alternative cleaners include: Bleach, Ammonia, Quat Disinfectant and Virox™. These cleaners must be used according to the manufacturer's instructions, and any deviation from the manufacturer's instructions may void the warranty.
- ✓ After cleaning, dampen a clean cloth with water to rinse the laminate surface, then wipe with a clean dry cloth. This step prevents cleaning agents from building up on the laminate surface, where moisture can cause them to reactivate and damage the surface.
- ✗ Avoid chemical or abrasive cleaning compounds.

Fabric

- ✓ Due to the unique properties of each upholstery material, we recommend contacting the manufacturer for safe and effective cleaning and maintenance instructions.

Maharam: 800.645.3943 or maharam.com

Momentum: 800.366.6839 or memosamples.com

DesignTex: 800.221.1540 or designtex.com

CF Stinson: 800.841.6279, cfstinson.com

Cleaning guidelines for other brands can be found in the Customer Service section of nemschoff.com.